

Table of contents

\\Physics

John Mc

Diffusion_QA

Diffusion_qa_qiba_v3

localizer
t1_vibe_fs_cor_p2_at_start
ep2d_diff_cor
localizer
ep2d_diff_ax
localizer
ep2d_diff_sag
t1_vibe_fs_sag_p2_at_end

\\Physics\John Mc\Diffusion_QA\Diffusion_qa_qiba_v3\localizer

TA: 0:12 PM: REF Voxel size: 0.5×0.5×7.0 mmPAT: Off Rel. SNR: 1.00 : fl

Properties

Prio recon	On
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	On
Load images to graphic segments	On
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice group	2
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
Slice group	3
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Coronal
Phase enc. dir.	R >> L
AutoAlign	---
Phase oversampling	0 %
FoV read	250 mm
FoV phase	100.0 %
Slice thickness	7.0 mm
TR	7.5 ms
TE	3.69 ms
Averages	2
Concatenations	3
Filter	Prescan Normalize, Elliptical filter
Coil elements	HEA;HEP

Contrast - Common

TR	7.5 ms
TE	3.69 ms
TD	0 ms
MTC	Off
Magn. preparation	None
Flip angle	20 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Contrast - Dynamic

Averages	2
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1

Contrast - Dynamic

Multiple series	Each measurement
-----------------	------------------

Resolution - Common

FoV read	250 mm
FoV phase	100.0 %
Slice thickness	7.0 mm
Base resolution	256
Phase resolution	91 %
Phase partial Fourier	Off
Interpolation	On

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	On

Geometry - Common

Slice group	1
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice group	2
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
Slice group	3
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Coronal
Phase enc. dir.	R >> L
FoV read	250 mm
FoV phase	100.0 %
Slice thickness	7.0 mm
TR	7.5 ms
Multi-slice mode	Sequential
Series	Interleaved
Concatenations	3

Geometry - AutoAlign

Slice group	1
Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice group	2
Position	L0.0 A20.0 H0.0 mm

Geometry - AutoAlign

Orientation	Transversal
Phase enc. dir.	A >> P
Slice group	3
Position	L0.0 A20.0 H0.0 mm
Orientation	Coronal
Phase enc. dir.	R >> L
AutoAlign	---
Initial Position	L0.0 A20.0 H0.0
L	0.0 mm
A	20.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Sagittal

Geometry - Saturation

Saturation mode	Standard
Fat suppr.	None
Water suppr.	None
Special sat.	None

System - Miscellaneous

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - AutoCoilSelect

System - Adjustments

B0 Shim mode	Tune up
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Slice-sel.

System - Tx/Rx

Frequency 1H	123.205026 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	7.5 ms
Concatenations	3
Segments	1

Physio - Cardiac

Tagging	None
Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	250 mm
FoV phase	100.0 %
Phase resolution	91 %

Physio - PACE

Resp. control	Off
Concatenations	3

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Soft Tissue

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

Inline - Composing

Distortion Corr.	Off
------------------	-----

Sequence - Part 1

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Allowed
Contrasts	1
Flow comp.	No
Multi-slice mode	Sequential
Bandwidth	320 Hz/Px

Sequence - Part 2

Segments	1
Acoustic noise reduction	None
RF pulse type	Fast
Gradient mode	Fast
Excitation	Slice-sel.
RF spoiling	On

Sequence - Assistant

Mode	Off
------	-----

Sequence - Assistant

Allowed delay	0 s
---------------	-----

\\Physics\John Mc\Diffusion_QA\Diffusion_qa_qiba_v3\11_vibe_fs_cor_p2_at_start

TA: 3:25 PM: REF Voxel size: 1.0×1.0×1.0 mmPAT: 2 Rel. SNR: 1.00 : fl

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	20 %
Position	R0.6 A1.1 H3.6 mm
Orientation	C > T-0.4
Phase enc. dir.	F >> H
AutoAlign	Head > Orbits
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	192
FoV read	256 mm
FoV phase	100.0 %
Slice thickness	1.0 mm
TR	7.58 ms
TE	2.88 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize, Elliptical filter
Coil elements	HEA;HEP

Contrast - Common

TR	7.58 ms
TE	2.88 ms
Flip angle	10.0 deg
Fat suppr.	None
Water suppr.	None
Dixon	Off
Dixon evaluation	Off

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	256 mm
FoV phase	100.0 %
Slice thickness	1.0 mm
Base resolution	256
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off
Slice partial Fourier	Off

Resolution - Common

Trajectory	Cartesian
View sharing	Off
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	24
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	On
POCS	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	20 %
Position	R0.6 A1.1 H3.6 mm
Orientation	C > T-0.4
Phase enc. dir.	F >> H
Slice oversampling	0.0 %
Slices per slab	192
FoV read	256 mm
FoV phase	100.0 %
Slice thickness	1.0 mm
TR	7.58 ms
Multi-slice mode	Sequential
Series	Ascending
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	R0.6 A1.1 H3.6 mm
Orientation	C > T-0.4
Phase enc. dir.	F >> H
AutoAlign	Head > Orbits
Initial Position	R0.6 A1.1 H3.6
R	0.6 mm
A	1.1 mm
H	3.6 mm
Initial Rotation	88.58 deg
Initial Orientation	C > T
C > T	-0.4
> S	0.0

Geometry - Saturation

Fat suppr.	None
Water suppr.	None
Dixon	Off

Geometry - Saturation

Dixon evaluation	Off
Special sat.	None

System - Miscellaneous

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	Head > Orbits
Coil Select Mode	Off - AutoCoilSelect

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	R0.6 A1.1 H3.6 mm
Orientation	C > T-0.4
Rotation	88.58 deg
F >> H	256 mm
R >> L	256 mm
A >> P	192 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Slab-sel.

System - Tx/Rx

Frequency 1H	123.205026 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

View sharing	Off
Flip angle	10.0 deg
Measurements	1
Burn time-to-center	Off
Temporal interpolation	1
3D centric reordering	Off
Time to center	103.9 s

Inline - Inline

Subtract	Off
Measurements	1

Inline - Inline

StdDev	Off
Liver registration	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Soft Tissue

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

Inline - Composing

Distortion Corr.	Off
------------------	-----

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	Off
Asymmetric echo	Weak
Contrasts	1
Optimization	None
Multi-slice mode	Sequential
Bandwidth	810 Hz/Px

Sequence - Part 2

RF pulse type	Normal
Gradient mode	Normal
Excitation	Slab-sel.
RF spoiling	On
Incr. Gradient spoiling	On

Sequence - Assistant

Mode	Off
Allowed delay	30 s

\\Physics\John Mc\Diffusion_QA\Diffusion_qa_qiba_v3\lep2d_diff_cor

TA: 3:54 PM: REF Voxel size: 1.7×1.7×4.0 mmPAT: 2 Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	35
Dist. factor	25 %
Position	L0.0 P0.0 H2.4 mm
Orientation	C > T-0.3
Phase enc. dir.	F >> H
AutoAlign	---
Phase oversampling	0 %
FoV read	220 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
TR	8000 ms
TE	81.0 ms
Averages	2
Concatenations	1
Filter	Raw filter, Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	8000 ms
TE	81.0 ms
MTC	Off
Magn. preparation	None
Fat suppr.	None

Contrast - Dynamic

Averages	2
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms

Resolution - Common

FoV read	220 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
Base resolution	130
Phase resolution	100 %
Phase partial Fourier	6/8
Interpolation	Off

Resolution - iPAT

Accel. mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	40

Resolution - iPAT

Reference scan mode	EPI/separate
---------------------	--------------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On
Dynamic Field Corr.	Off

Resolution - Filter Rawdata

Raw filter	On
Elliptical filter	Off

Geometry - Common

Slice group	1
Slices	35
Dist. factor	25 %
Position	L0.0 P0.0 H2.4 mm
Orientation	C > T-0.3
Phase enc. dir.	F >> H
FoV read	220 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
TR	8000 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	L0.0 P0.0 H2.4 mm
Orientation	C > T-0.3
Phase enc. dir.	F >> H
AutoAlign	---
Initial Position	L0.0 P0.0 H2.4
L	0.0 mm
P	0.0 mm
H	2.4 mm
Initial Rotation	90.00 deg
Initial Orientation	C > T
C > T	-0.3
> S	0.0

Geometry - Saturation

Fat suppr.	None
Special sat.	None

Geometry - Navigator**System - Miscellaneous**

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	On - AutoCoilSelect

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

! Position	Isocenter
! Orientation	Coronal
! Rotation	0.00 deg
! R >> L	216 mm
! F >> H	216 mm
! A >> P	35 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.205026 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	8000 ms
Concatenations	1

Physio - PACE

Resp. control	Off
Concatenations	1

Diff - Neuro

Diffusion mode	3-Scan Trace
Diff. directions	3
Diffusion Scheme	Bipolar
Diff. weightings	5
b-value 1	0 s/mm ²
b-value 2	500 s/mm ²
b-value 3	1000 s/mm ²
b-value 4	1500 s/mm ²
b-value 5	2000 s/mm ²
b-value 1	2
b-value 2	2
b-value 3	2
b-value 4	2
b-value 5	2
Diff. weighted images	Off
Trace weighted images	On
ADC maps	On
FA maps	Off
Mosaic	Off
Tensor	Off
Noise level	40

Diff - Body

Diffusion mode	3-Scan Trace
----------------	--------------

Diff - Body

Diff. directions	3
Diffusion Scheme	Bipolar
Diff. weightings	5
b-value 1	0 s/mm ²
b-value 2	500 s/mm ²
b-value 3	1000 s/mm ²
b-value 4	1500 s/mm ²
b-value 5	2000 s/mm ²
b-value 1	2
b-value 2	2
b-value 3	2
b-value 4	2
b-value 5	2
Diff. weighted images	Off
Trace weighted images	On
ADC maps	On
Exponential ADC Maps	Off
FA maps	Off
Invert Gray Scale	Off
Calculated Image	Off
b-Value >=	0 s/mm ²
Noise level	40

Diff - Composing

Distortion Corr.	Off
------------------	-----

Sequence - Part 1

Introduction	On
Optimization	None
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.93 ms
Bandwidth	1202 Hz/Px

Sequence - Part 2

EPI factor	130
RF pulse type	Normal
Gradient mode	Performance*
Excitation	Standard

Sequence - pTX Pulses

\\Physics\John Mc\Diffusion_QA\Diffusion_qa_qiba_v3\localizer

TA: 0:12 PM: REF Voxel size: 0.5×0.5×7.0 mmPAT: Off Rel. SNR: 1.00 : fl

Properties

Prio recon	On
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	On
Load images to graphic segments	On
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice group	2
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
Slice group	3
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Coronal
Phase enc. dir.	R >> L
AutoAlign	---
Phase oversampling	0 %
FoV read	250 mm
FoV phase	100.0 %
Slice thickness	7.0 mm
TR	7.5 ms
TE	3.69 ms
Averages	2
Concatenations	3
Filter	Prescan Normalize, Elliptical filter
Coil elements	HEA;HEP

Contrast - Common

TR	7.5 ms
TE	3.69 ms
TD	0 ms
MTC	Off
Magn. preparation	None
Flip angle	20 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Contrast - Dynamic

Averages	2
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1

Contrast - Dynamic

Multiple series	Each measurement
-----------------	------------------

Resolution - Common

FoV read	250 mm
FoV phase	100.0 %
Slice thickness	7.0 mm
Base resolution	256
Phase resolution	91 %
Phase partial Fourier	Off
Interpolation	On

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	On

Geometry - Common

Slice group	1
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice group	2
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
Slice group	3
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Coronal
Phase enc. dir.	R >> L
FoV read	250 mm
FoV phase	100.0 %
Slice thickness	7.0 mm
TR	7.5 ms
Multi-slice mode	Sequential
Series	Interleaved
Concatenations	3

Geometry - AutoAlign

Slice group	1
Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice group	2
Position	L0.0 A20.0 H0.0 mm

Geometry - AutoAlign

Orientation	Transversal
Phase enc. dir.	A >> P
Slice group	3
Position	L0.0 A20.0 H0.0 mm
Orientation	Coronal
Phase enc. dir.	R >> L
AutoAlign	---
Initial Position	L0.0 A20.0 H0.0
L	0.0 mm
A	20.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Sagittal

Geometry - Saturation

Saturation mode	Standard
Fat suppr.	None
Water suppr.	None
Special sat.	None

System - Miscellaneous

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - AutoCoilSelect

System - Adjustments

B0 Shim mode	Tune up
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Slice-sel.

System - Tx/Rx

Frequency 1H	123.205026 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	7.5 ms
Concatenations	3
Segments	1

Physio - Cardiac

Tagging	None
Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	250 mm
FoV phase	100.0 %
Phase resolution	91 %

Physio - PACE

Resp. control	Off
Concatenations	3

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Soft Tissue

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

Inline - Composing

Distortion Corr.	Off
------------------	-----

Sequence - Part 1

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Allowed
Contrasts	1
Flow comp.	No
Multi-slice mode	Sequential
Bandwidth	320 Hz/Px

Sequence - Part 2

Segments	1
Acoustic noise reduction	None
RF pulse type	Fast
Gradient mode	Fast
Excitation	Slice-sel.
RF spoiling	On

Sequence - Assistant

Mode	Off
------	-----

Sequence - Assistant

Allowed delay	0 s
---------------	-----

\\Physics\John Mc\Diffusion_QA\Diffusion_qa_qiba_v3lep2d_diff_ax

TA: 3:54 PM: REF Voxel size: 1.7×1.7×4.0 mmPAT: 2 Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	35
Dist. factor	25 %
Position	L0.0 P1.8 F4.8 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Phase oversampling	0 %
FoV read	220 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
TR	8000 ms
TE	81.0 ms
Averages	2
Concatenations	1
Filter	Raw filter, Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	8000 ms
TE	81.0 ms
MTC	Off
Magn. preparation	None
Fat suppr.	None

Contrast - Dynamic

Averages	2
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms

Resolution - Common

FoV read	220 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
Base resolution	130
Phase resolution	100 %
Phase partial Fourier	6/8
Interpolation	Off

Resolution - iPAT

Accel. mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	40

Resolution - iPAT

Reference scan mode	EPI/separate
---------------------	--------------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On
Dynamic Field Corr.	Off

Resolution - Filter Rawdata

Raw filter	On
Elliptical filter	Off

Geometry - Common

Slice group	1
Slices	35
Dist. factor	25 %
Position	L0.0 P1.8 F4.8 mm
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	220 mm
FoV phase	100.0 %
Slice thickness	4.0 mm
TR	8000 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	L0.0 P1.8 F4.8 mm
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	---
Initial Position	L0.0 P1.8 F4.8
L	0.0 mm
P	1.8 mm
F	4.8 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	None
Special sat.	None

Geometry - Navigator**System - Miscellaneous**

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	On - AutoCoilSelect

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

! Position	Isocenter
! Orientation	Coronal
! Rotation	0.00 deg
! R >> L	216 mm
! F >> H	216 mm
! A >> P	35 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.205026 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	8000 ms
Concatenations	1

Physio - PACE

Resp. control	Off
Concatenations	1

Diff - Neuro

Diffusion mode	3-Scan Trace
Diff. directions	3
Diffusion Scheme	Bipolar
Diff. weightings	5
b-value 1	0 s/mm ²
b-value 2	500 s/mm ²
b-value 3	1000 s/mm ²
b-value 4	1500 s/mm ²
b-value 5	2000 s/mm ²
b-value 1	2
b-value 2	2
b-value 3	2
b-value 4	2
b-value 5	2
Diff. weighted images	Off
Trace weighted images	On
ADC maps	On
FA maps	Off
Mosaic	Off
Tensor	Off
Noise level	40

Diff - Body

Diffusion mode	3-Scan Trace
----------------	--------------

Diff - Body

Diff. directions	3
Diffusion Scheme	Bipolar
Diff. weightings	5
b-value 1	0 s/mm ²
b-value 2	500 s/mm ²
b-value 3	1000 s/mm ²
b-value 4	1500 s/mm ²
b-value 5	2000 s/mm ²
b-value 1	2
b-value 2	2
b-value 3	2
b-value 4	2
b-value 5	2
Diff. weighted images	Off
Trace weighted images	On
ADC maps	On
Exponential ADC Maps	Off
FA maps	Off
Invert Gray Scale	Off
Calculated Image	Off
b-Value >=	0 s/mm ²
Noise level	40

Diff - Composing

Distortion Corr.	Off
------------------	-----

Sequence - Part 1

Introduction	On
Optimization	None
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.93 ms
Bandwidth	1202 Hz/Px

Sequence - Part 2

EPI factor	130
RF pulse type	Normal
Gradient mode	Performance*
Excitation	Standard

Sequence - pTX Pulses

\\Physics\John Mc\Diffusion_QA\Diffusion_qa_qiba_v3\localizer

TA: 0:12 PM: REF Voxel size: 0.5×0.5×7.0 mmPAT: Off Rel. SNR: 1.00 : fl

Properties

Prio recon	On
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	On
Load images to graphic segments	On
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice group	2
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
Slice group	3
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Coronal
Phase enc. dir.	R >> L
AutoAlign	---
Phase oversampling	0 %
FoV read	250 mm
FoV phase	100.0 %
Slice thickness	7.0 mm
TR	7.5 ms
TE	3.69 ms
Averages	2
Concatenations	3
Filter	Prescan Normalize, Elliptical filter
Coil elements	HEA;HEP

Contrast - Common

TR	7.5 ms
TE	3.69 ms
TD	0 ms
MTC	Off
Magn. preparation	None
Flip angle	20 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Contrast - Dynamic

Averages	2
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1

Contrast - Dynamic

Multiple series	Each measurement
-----------------	------------------

Resolution - Common

FoV read	250 mm
FoV phase	100.0 %
Slice thickness	7.0 mm
Base resolution	256
Phase resolution	91 %
Phase partial Fourier	Off
Interpolation	On

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	On

Geometry - Common

Slice group	1
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice group	2
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Transversal
Phase enc. dir.	A >> P
Slice group	3
Slices	1
Dist. factor	20 %
Position	L0.0 A20.0 H0.0 mm
Orientation	Coronal
Phase enc. dir.	R >> L
FoV read	250 mm
FoV phase	100.0 %
Slice thickness	7.0 mm
TR	7.5 ms
Multi-slice mode	Sequential
Series	Interleaved
Concatenations	3

Geometry - AutoAlign

Slice group	1
Position	L0.0 A20.0 H0.0 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice group	2
Position	L0.0 A20.0 H0.0 mm

Geometry - AutoAlign

Orientation	Transversal
Phase enc. dir.	A >> P
Slice group	3
Position	L0.0 A20.0 H0.0 mm
Orientation	Coronal
Phase enc. dir.	R >> L
AutoAlign	---
Initial Position	L0.0 A20.0 H0.0
L	0.0 mm
A	20.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Sagittal

Geometry - Saturation

Saturation mode	Standard
Fat suppr.	None
Water suppr.	None
Special sat.	None

System - Miscellaneous

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - AutoCoilSelect

System - Adjustments

B0 Shim mode	Tune up
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Slice-sel.

System - Tx/Rx

Frequency 1H	123.205026 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	7.5 ms
Concatenations	3
Segments	1

Physio - Cardiac

Tagging	None
Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	250 mm
FoV phase	100.0 %
Phase resolution	91 %

Physio - PACE

Resp. control	Off
Concatenations	3

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Soft Tissue

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

Inline - Composing

Distortion Corr.	Off
------------------	-----

Sequence - Part 1

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Allowed
Contrasts	1
Flow comp.	No
Multi-slice mode	Sequential
Bandwidth	320 Hz/Px

Sequence - Part 2

Segments	1
Acoustic noise reduction	None
RF pulse type	Fast
Gradient mode	Fast
Excitation	Slice-sel.
RF spoiling	On

Sequence - Assistant

Mode	Off
------	-----

Sequence - Assistant

Allowed delay	0 s
---------------	-----

\\Physics\John Mc\Diffusion_QA\Diffusion_qa_qiba_v3\ep2d_diff_sag

TA: 3:54 PM: REF Voxel size: 1.7×1.7×4.0 mmPAT: 2 Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	35
Dist. factor	25 %
Position	L0.0 A4.2 H2.4 mm
Orientation	Sagittal
Phase enc. dir.	H >> F
AutoAlign	---
Phase oversampling	0 %
FoV read	220 mm
FoV phase	101.5 %
Slice thickness	4.0 mm
TR	8000 ms
TE	81.0 ms
Averages	2
Concatenations	1
Filter	Raw filter, Prescan Normalize
Coil elements	HEA;HEP

Contrast - Common

TR	8000 ms
TE	81.0 ms
MTC	Off
Magn. preparation	None
Fat suppr.	None

Contrast - Dynamic

Averages	2
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Delay in TR	0 ms

Resolution - Common

FoV read	220 mm
FoV phase	101.5 %
Slice thickness	4.0 mm
Base resolution	130
Phase resolution	100 %
Phase partial Fourier	6/8
Interpolation	Off

Resolution - iPAT

Accel. mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	40

Resolution - iPAT

Reference scan mode	EPI/separate
---------------------	--------------

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	On
Dynamic Field Corr.	Off

Resolution - Filter Rawdata

Raw filter	On
Elliptical filter	Off

Geometry - Common

Slice group	1
Slices	35
Dist. factor	25 %
Position	L0.0 A4.2 H2.4 mm
Orientation	Sagittal
Phase enc. dir.	H >> F
FoV read	220 mm
FoV phase	101.5 %
Slice thickness	4.0 mm
TR	8000 ms
Multi-slice mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice group	1
Position	L0.0 A4.2 H2.4 mm
Orientation	Sagittal
Phase enc. dir.	H >> F
AutoAlign	---
Initial Position	L0.0 A4.2 H2.4
L	0.0 mm
A	4.2 mm
H	2.4 mm
Initial Rotation	90.00 deg
Initial Orientation	Sagittal

Geometry - Saturation

Fat suppr.	None
Special sat.	None

Geometry - Navigator**System - Miscellaneous**

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	On - AutoCoilSelect

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	L0.0 A4.2 H2.4 mm
Orientation	Sagittal
Rotation	180.00 deg
A >> P	220 mm
F >> H	224 mm
R >> L	174 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	123.205026 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	8000 ms
Concatenations	1

Physio - PACE

Resp. control	Off
Concatenations	1

Diff - Neuro

Diffusion mode	3-Scan Trace
Diff. directions	3
Diffusion Scheme	Bipolar
Diff. weightings	5
b-value 1	0 s/mm ²
b-value 2	500 s/mm ²
b-value 3	1000 s/mm ²
b-value 4	1500 s/mm ²
b-value 5	2000 s/mm ²
b-value 1	2
b-value 2	2
b-value 3	2
b-value 4	2
b-value 5	2
Diff. weighted images	Off
Trace weighted images	On
ADC maps	On
FA maps	Off
Mosaic	Off
Tensor	Off
Noise level	40

Diff - Body

Diffusion mode	3-Scan Trace
----------------	--------------

Diff - Body

Diff. directions	3
Diffusion Scheme	Bipolar
Diff. weightings	5
b-value 1	0 s/mm ²
b-value 2	500 s/mm ²
b-value 3	1000 s/mm ²
b-value 4	1500 s/mm ²
b-value 5	2000 s/mm ²
b-value 1	2
b-value 2	2
b-value 3	2
b-value 4	2
b-value 5	2
Diff. weighted images	Off
Trace weighted images	On
ADC maps	On
Exponential ADC Maps	Off
FA maps	Off
Invert Gray Scale	Off
Calculated Image	Off
b-Value >=	0 s/mm ²
Noise level	40

Diff - Composing

Distortion Corr.	Off
------------------	-----

Sequence - Part 1

Introduction	On
Optimization	None
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.93 ms
Bandwidth	1202 Hz/Px

Sequence - Part 2

EPI factor	132
RF pulse type	Normal
Gradient mode	Performance*
Excitation	Standard

Sequence - pTX Pulses

\\Physics\John Mc\Diffusion_QA\Diffusion_qa_qiba_v3\t1_vibe_fs_sag_p2_at_end

TA: 3:25 PM: REF Voxel size: 1.0×1.0×1.0 mmPAT: 2 Rel. SNR: 1.00 : fl

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	20 %
Position	R0.6 A1.1 H3.6 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	Head > Orbits
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	192
FoV read	256 mm
FoV phase	100.0 %
Slice thickness	1.0 mm
TR	7.58 ms
TE	2.88 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize, Elliptical filter
Coil elements	HEA;HEP

Contrast - Common

TR	7.58 ms
TE	2.88 ms
Flip angle	10.0 deg
Fat suppr.	None
Water suppr.	None
Dixon	Off
Dixon evaluation	Off

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	256 mm
FoV phase	100.0 %
Slice thickness	1.0 mm
Base resolution	256
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off
Slice partial Fourier	Off

Resolution - Common

Trajectory	Cartesian
View sharing	Off
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	24
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	On
Unfiltered images	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	On
POCS	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	20 %
Position	R0.6 A1.1 H3.6 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice oversampling	0.0 %
Slices per slab	192
FoV read	256 mm
FoV phase	100.0 %
Slice thickness	1.0 mm
TR	7.58 ms
Multi-slice mode	Sequential
Series	Ascending
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	R0.6 A1.1 H3.6 mm
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	Head > Orbits
Initial Position	R0.6 A1.1 H3.6
R	0.6 mm
A	1.1 mm
H	3.6 mm
Initial Rotation	0.00 deg
Initial Orientation	Sagittal

Geometry - Saturation

Fat suppr.	None
Water suppr.	None
Dixon	Off
Dixon evaluation	Off
Special sat.	None

System - Miscellaneous

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	Head > Orbits
Coil Select Mode	Off - AutoCoilSelect

System - Adjustments

B0 Shim mode	Standard
B1 Shim mode	TrueForm
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	R0.6 A1.1 H3.6 mm
Orientation	Sagittal
Rotation	0.00 deg
A >> P	256 mm
F >> H	256 mm
R >> L	192 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Slab-sel.

System - Tx/Rx

Frequency 1H	123.205026 MHz
Correction factor	1
Gain	Low
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

View sharing	Off
Flip angle	10.0 deg
Measurements	1
Burn time-to-center	Off
Temporal interpolation	1
3D centric reordering	Off
Time to center	103.9 s

Inline - Inline

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Soft Tissue

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

Inline - Composing

Distortion Corr.	Off
------------------	-----

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	Off
Asymmetric echo	Weak
Contrasts	1
Optimization	None
Multi-slice mode	Sequential
Bandwidth	810 Hz/Px

Sequence - Part 2

RF pulse type	Normal
Gradient mode	Normal
Excitation	Slab-sel.
RF spoiling	On
Incr. Gradient spoiling	On

Sequence - Assistant

Mode	Off
Allowed delay	30 s