

Table of contents

\ZZ_DWI_QIBA

DWI_QIBA

QA_MSC

QA_200622

localizer
QIBA_qa_vibe_cor_iso
QIBA_diff_qa_cor
localizer
QIBA_diff_qa_ax
localizer
qiba_diff_qa_sag
QIBA_qa_vibe_sag_iso

\ZZ_DWI_QIBA\DWI_QIBA\QA_MSC\QA_200622\localizer

TA: 31 sec Coil Selection: Auto Voxel Size: 1.2×1.2×10.0 mm³ Acc.: None Rel. SNR: 1.00**Properties**

Start measurement without further preparation	Off
Wait for User to Start	Off
Start measurements	Single Measurement
Prio Recon	Off
Auto Open Inline Display	Off
Auto Close Inline Display	Off
Load Images to MR View&GO	On
Auto Store Images	On
Load Images to Stamp Segments	On
Load Images to Graphic Segments	On
Graphic segment	Default
Inline Movie	Off

Routine

Slice Group	1
Slices	1
Distance Factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Slice Group	2
Slices	1
Distance Factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Slice Group	3
Slices	1
Distance Factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Phase Oversampling	0 %
FoV Read	300 mm
FoV Phase	100.0 %
Slice Thickness	10.0 mm
TR	13.0 ms
TE	7.30 ms
Averages	1
Concatenations	18
AutoAlign	---

Contrast - Common

TR	13.0 ms
TE	7.30 ms
TD	0.00 ms
MTC	Off
Magn. Preparation	None
Flip Angle	40 deg
Fat-Water Contrast	Standard
Dark Blood	Off
Contrasts	1
Reconstruction	Magnitude

Contrast - Dynamic

Dynamic Mode	Standard
Measurements	1
Multiple Series	Each Measurement

Resolution - Common

FoV Read	300 mm
FoV Phase	100.0 %
Slice Thickness	10.0 mm
Base Resolution	256
Phase Resolution	50 %
Interpolation	Off

Resolution - Acceleration

Acceleration mode	None
Phase Partial Fourier	Off
Asymmetric Echo	Off

Resolution - Filter

Raw Filter	Off
Elliptical Filter	Off
Distortion Correction	2D
Normalize	Off
Image Filter	Off

Geometry - Common

Slice Group	1
Slices	1
Distance Factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Slice Group	2
Slices	1
Distance Factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Slice Group	3
Slices	1
Distance Factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Phase Oversampling	0 %
FoV Read	300 mm
FoV Phase	100.0 %
Slice Thickness	10.0 mm
TR	13.0 ms
Multi-Slice Mode	Sequential
Series	Interleaved
Concatenations	18

Geometry - AutoAlign

Slice Group	1
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Slice Group	2
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Slice Group	3
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P

Geometry - AutoAlign

AutoAlign	---
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Orientation	Sagittal
Initial Rotation	0.00 deg

Geometry - Saturation

Saturation Mode	Standard
Special Saturation	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table Position	0 mm
Table Position	H
Inline Composing	Off

System - Miscellaneous

Coil Selection	Auto Coil Select
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combination	Adaptive Combine
Matrix Optimization	Off
Coil Focus	Flat

System - Adjustments

Adjustment Strategy	Standard
B0 Shim	Tune up
CoilShim	Off
Adjustment Tolerance	Auto
Adjust with Body Coil	Off
Confirm Frequency	Never
Assume Silicone	Off

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

System - Tx/Rx

Frequency 1H	63.680140 MHz
? Ref. Amplitude 1H	0.000 V
Reset	Off
Image Scaling	1.000

Physio - Signal

1st Signal/Mode	None
TR	13.0 ms
Segments	1
Concatenations	18

Physio - Cardiac

Tagging	None
Fat-Water Contrast	Standard
Magn. Preparation	None
Dark Blood	Off

Physio - Cardiac

FoV Read	300 mm
FoV Phase	100.0 %
Phase Resolution	50 %
Dynamic Mode	Standard

Physio - PACE

Resp. Control	Off
Concatenations	18

Inline - Liver

Liver Registration	Off
Save Original Images	On

Inline - Subtraction

Subtract	Off
Measurements	1
StdDev	Off
Save Original Images	On

Inline - Cardiac

Magn. Preparation	None
Save Original Images	On
Contrasts	1
TE	7.30 ms
TR	13.0 ms

Inline - MIP

MIP Sag	Off
MIP Cor	Off
MIP Tra	Off
MIP Time	Off
Radial MIP	Off
Save Original Images	On
MPR Sag	Off
MPR Cor	Off
MPR Tra	Off

Inline - Soft Tissue

Wash-in	Off
Wash-out	Off
TTP	Off
PEI	Off
MIP Time	Off
Measurements	1

Inline - Composing

Inline Composing	Off
------------------	-----

Sequence - Part 1

Sequence Name	fl_rr
Dimension	2D
Excitation	Slice-sel.
RF Pulse Type	Normal
Gradient Mode	Whisper
Flow Compensation	Read
Bandwidth	210 Hz/Px
Asymmetric Echo	Off
Segments	1

Sequence - Part 2

Introduction	On
RF Spoiling	On
Acoustic noise reduction	Off

Sequence - Assistant

SAR Assistant	Off
---------------	-----

\\ZZ_DWI_QIBA\DWI_QIBA\QA_MSC\QA_200622\QIBA_qa_vibe_cor_iso

TA: 2:43 min Coil Selection: Auto Voxel Size: 1.0×1.0×1.0 mm³ Acc.: 2 Rel. SNR: 1.00**Properties**

Start measurement without further preparation	On
Wait for User to Start	Off
Start measurements	Single Measurement
Prio Recon	Off
Auto Open Inline Display	Off
Auto Close Inline Display	Off
Load Images to MR View&GO	On
Auto Store Images	On
Load Images to Stamp Segments	On
Load Images to Graphic Segments	Off
Graphic segment	Default
Inline Movie	Off

Routine

Slab Group	1
Slabs	1
Distance Factor	20 %
Position	R1.4 P18.6 H0.7 mm
Orientation	C > T-1.4
Phase Encoding Dir.	F >> H
Slices per Slab	192
Phase Oversampling	0 %
Slice Oversampling	0.0 %
FoV Read	250 mm
FoV Phase	100.0 %
Slice Thickness	1.0 mm
TR	6.0 ms
TE	2.38 ms
Averages	1
Concatenations	1
AutoAlign	Head > Orbits

Contrast - Common

TR	6.0 ms
TE	2.38 ms
Flip Angle	10 deg
Fat-Water Contrast	Standard
Contrasts	1
Reconstruction	Magnitude

Contrast - Dynamic

Dynamic Mode	Standard
Temporal Interpolation	1
Measurements	1
Multiple Series	Off
3D Reordering	Standard
Time to Center	82.6 s
Burn Time to Center	Off

Resolution - Common

FoV Read	250 mm
FoV Phase	100.0 %
Slice Thickness	1.0 mm
Base Resolution	256
Phase Resolution	100 %
Slice Resolution	100 %
Trajectory	Cartesian
Interpolation	Off

Resolution - Acceleration

Acceleration mode	GRAPPA
Total Factor	2
Reference Scans	Integrated
Acceleration Factor PE	2
Reference Lines PE	24
Acceleration Factor 3D	1
Phase Partial Fourier	Off
Slice Partial Fourier	Off
Asymmetric Echo	Weak
Elliptical Scanning	Off

Resolution - Filter

Raw Filter	Off
Elliptical Filter	On
POCS	Off
Distortion Correction	2D
Normalize	Prescan
Noise Masking	Off
Image Filter	Off

Geometry - Common

Slab Group	1
Slabs	1
Distance Factor	20 %
Position	R1.4 P18.6 H0.7 mm
Orientation	C > T-1.4
Phase Encoding Dir.	F >> H
Slices per Slab	192
Phase Oversampling	0 %
Slice Oversampling	0.0 %
FoV Read	250 mm
FoV Phase	100.0 %
Slice Thickness	1.0 mm
TR	6.0 ms
Series	Ascending
Concatenations	1

Geometry - AutoAlign

Slab Group	1
Position	R1.4 P18.6 H0.7 mm
Orientation	C > T-1.4
Phase Encoding Dir.	F >> H
AutoAlign	Head > Orbits
Initial Position	R1.4 P18.6 H0.7
R	1.4 mm
P	18.6 mm
H	0.7 mm
Initial Orientation	C > T
C > T	-1.40
> S	0.00
Initial Rotation	90.00 deg

Geometry - Saturation

Special Saturation	None
--------------------	------

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table Position	0 mm
Table Position	H
Inline Composing	Off

System - Miscellaneous

Coil Selection	Auto Coil Select
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combination	Adaptive Combine
Matrix Optimization	Off
Coil Focus	Flat

System - Adjustments

Adjustment Strategy	Standard
B0 Shim	Standard
CoilShim	Off
Adjustment Tolerance	Auto
Adjust with Body Coil	Off
Confirm Frequency	Never
Assume Silicone	Off

System - Adjust Volume

Position	R1.4 P18.6 H0.7 mm
Orientation	C > T-1.4
Rotation	90.00 deg
F >> H	250 mm
R >> L	250 mm
A >> P	192 mm
Reset	Off

System - Tx/Rx

Frequency 1H	63.680140 MHz
? Ref. Amplitude 1H	0.000 V
Reset	Off
Image Scaling	1.000

Physio - PACE

Resp. Control	Off
Concatenations	1

Inline - Liver

Liver Registration	Off
Save Original Images	On

Inline - Subtraction

Subtract	Off
Measurements	1
StdDev	Off
Save Original Images	On

Inline - Cardiac

Save Original Images	On
Contrasts	1
TE	2.38 ms
TR	6.0 ms

Inline - MIP

MIP Sag	Off
MIP Cor	Off
MIP Tra	Off
MIP Time	Off
Radial MIP	Off
Save Original Images	On
MPR Sag	Off
MPR Cor	Off
MPR Tra	Off

Inline - Soft Tissue

Wash-in	Off
Wash-out	Off
TTP	Off
PEI	Off
MIP Time	Off
Measurements	1

Inline - Composing

Inline Composing	Off
------------------	-----

Sequence - Part 1

Sequence Name	fl
Dimension	3D
Excitation	Slab-sel.
RF Pulse Type	Normal
Gradient Mode	Normal
Bandwidth	810 Hz/Px
Asymmetric Echo	Weak
Optimization	None

Sequence - Part 2

Introduction	On
RF Spoiling	On
Incr. Gradient Spoiling	On
Breast Application	Off

Sequence - Assistant

SAR Assistant	Off
Optimization	None

\\ZZ_DWI_QIBA\DWI_QIBA\QA_MSC\QA_200622\QIBA_diff_qa_cor

TA: 3:54 min Coil Selection: Auto Voxel Size: 1.7×1.7×4.0 mm³ Acc:: 2 Rel. SNR: 1.00**Properties**

Start measurement without further preparation	Off
Wait for User to Start	Off
Start measurements	Single Measurement
Prio Recon	Off
Auto Open Inline Display	Off
Auto Close Inline Display	Off
Load Images to MR View&GO	On
Auto Store Images	On
Load Images to Stamp Segments	Off
Load Images to Graphic Segments	Off
Graphic segment	Default
Inline Movie	Off

Routine

Slice Group	1
Slices	35
Distance Factor	25 %
Position	L0.0 P18.6 H0.6 mm
Orientation	C > T-0.7 > S-0.6
Phase Encoding Dir.	F >> H
Phase Oversampling	0 %
FoV Read	220 mm
FoV Phase	100.0 %
Slice Thickness	4.0 mm
TR	8000.0 ms
TE	96.00 ms
Concatenations	1
AutoAlign	---

Contrast - Common

TR	8000.0 ms
TE	96.00 ms
MTC	Off
Magn. Preparation	None
Fat-Water Contrast	Standard
Reconstruction	Magnitude

Contrast - Dynamic

Dynamic Mode	Standard
Delay in TR	0.00 ms

Resolution - Common

FoV Read	220 mm
FoV Phase	100.0 %
Slice Thickness	4.0 mm
Base Resolution	130
Phase Resolution	100 %
Interpolation	Off

Resolution - Acceleration

Acceleration mode	GRAPPA
Reference Scans	EPI/Separate
Acceleration Factor PE	2
Reference Lines PE	40
Phase Partial Fourier	6/8

Resolution - Filter

Raw Filter	On
Elliptical Filter	Off

Resolution - Filter

Distortion Correction	2D
Normalize	Prescan
Noise Masking	Off

Geometry - Common

Slice Group	1
Slices	35
Distance Factor	25 %
Position	L0.0 P18.6 H0.6 mm
Orientation	C > T-0.7 > S-0.6
Phase Encoding Dir.	F >> H
Phase Oversampling	0 %
FoV Read	220 mm
FoV Phase	100.0 %
Slice Thickness	4.0 mm
TR	8000.0 ms
Multi-Slice Mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice Group	1
Position	L0.0 P18.6 H0.6 mm
Orientation	C > T-0.7 > S-0.6
Phase Encoding Dir.	F >> H
AutoAlign	---
Initial Position	L0.0 P18.6 H0.6
L	0.0 mm
P	18.6 mm
H	0.6 mm
Initial Orientation	C > T
C > T	-0.70
> S	-0.60
Initial Rotation	90.00 deg

Geometry - Navigator**Geometry - Saturation**

Special Saturation	None
--------------------	------

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table Position	0 mm
Table Position	H
Inline Composing	Off

System - Miscellaneous

Coil Selection	Auto Coil Select
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combination	Adaptive Combine
Matrix Optimization	Off
Coil Focus	Flat

System - Adjustments

Adjustment Strategy	Standard
B0 Shim	Standard
CoilShim	Off

System - Adjustments

Adjustment Tolerance	Auto
Adjust with Body Coil	Off
Confirm Frequency	Never
Assume Silicone	Off

System - Adjust Volume

Position	L0.0 P18.6 H0.6 mm
Orientation	C > T-0.7 > S-0.6
Rotation	90.00 deg
F >> H	220 mm
R >> L	220 mm
A >> P	174 mm
Reset	Off

System - Tx/Rx

Frequency 1H	63.680140 MHz
? Ref. Amplitude 1H	0.000 V
Reset	Off
Image Scaling	1.000

Physio - Signal

1st Signal/Mode	None
TR	8000.0 ms
Concatenations	1

Physio - PACE

Resp. Control	Off
Concatenations	1

Diff

Diffusion Mode	3-Scan Trace
Diff. Directions	3
Diffusion Scheme	Bipolar
Diff. Weightings	5
b-value 1	0 s/mm ²
b-value 2	500 s/mm ²
b-value 3	1000 s/mm ²
b-value 4	1500 s/mm ²
b-value 5	2000 s/mm ²
Averages 1	2
Averages 2	2
Averages 3	2
Averages 4	2
Averages 5	2
Dynamic Field Correction	Off
Invert Gray Scale	Off
Diff. Weighted Images	Off
Trace Weighted Images	On
Tensor	Off
FA Maps	Off
ADC Maps	On
Exponential ADC Maps	Off
b-value >=	0 s/mm ²
ADC Noise Threshold	40
Noise Masking	Off
Calculated Image	Off

Sequence - Part 1

Sequence Name	epse
Excitation	Standard
RF Pulse Type	Normal
Gradient Mode	Fast
Bandwidth	1132 Hz/Px
Echo Spacing	0.97 ms

Sequence - Part 1

Free Echo Spacing	Off
Optimization	None
EPI Factor	130

Sequence - Part 2

Introduction	On
Phase Correction	Internal

\ZZ_DWI_QIBA\DWI_QIBA\QA_MSC\QA_200622\localizer

TA: 31 sec Coil Selection: Auto Voxel Size: 1.2×1.2×10.0 mm³ Acc.: None Rel. SNR: 1.00**Properties**

Start measurement without further preparation	Off
Wait for User to Start	Off
Start measurements	Single Measurement
Prio Recon	Off
Auto Open Inline Display	Off
Auto Close Inline Display	Off
Load Images to MR View&GO	On
Auto Store Images	On
Load Images to Stamp Segments	On
Load Images to Graphic Segments	On
Graphic segment	Default
Inline Movie	Off

Routine

Slice Group	1
Slices	1
Distance Factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Slice Group	2
Slices	1
Distance Factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Slice Group	3
Slices	1
Distance Factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Phase Oversampling	0 %
FoV Read	300 mm
FoV Phase	100.0 %
Slice Thickness	10.0 mm
TR	13.0 ms
TE	7.30 ms
Averages	1
Concatenations	18
AutoAlign	---

Contrast - Common

TR	13.0 ms
TE	7.30 ms
TD	0.00 ms
MTC	Off
Magn. Preparation	None
Flip Angle	40 deg
Fat-Water Contrast	Standard
Dark Blood	Off
Contrasts	1
Reconstruction	Magnitude

Contrast - Dynamic

Dynamic Mode	Standard
Measurements	1
Multiple Series	Each Measurement

Resolution - Common

FoV Read	300 mm
FoV Phase	100.0 %
Slice Thickness	10.0 mm
Base Resolution	256
Phase Resolution	50 %
Interpolation	Off

Resolution - Acceleration

Acceleration mode	None
Phase Partial Fourier	Off
Asymmetric Echo	Off

Resolution - Filter

Raw Filter	Off
Elliptical Filter	Off
Distortion Correction	2D
Normalize	Off
Image Filter	Off

Geometry - Common

Slice Group	1
Slices	1
Distance Factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Slice Group	2
Slices	1
Distance Factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Slice Group	3
Slices	1
Distance Factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Phase Oversampling	0 %
FoV Read	300 mm
FoV Phase	100.0 %
Slice Thickness	10.0 mm
TR	13.0 ms
Multi-Slice Mode	Sequential
Series	Interleaved
Concatenations	18

Geometry - AutoAlign

Slice Group	1
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Slice Group	2
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Slice Group	3
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P

Geometry - AutoAlign

AutoAlign	---
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Orientation	Sagittal
Initial Rotation	0.00 deg

Geometry - Saturation

Saturation Mode	Standard
Special Saturation	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table Position	0 mm
Table Position	H
Inline Composing	Off

System - Miscellaneous

Coil Selection	Auto Coil Select
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combination	Adaptive Combine
Matrix Optimization	Off
Coil Focus	Flat

System - Adjustments

Adjustment Strategy	Standard
B0 Shim	Tune up
CoilShim	Off
Adjustment Tolerance	Auto
Adjust with Body Coil	Off
Confirm Frequency	Never
Assume Silicone	Off

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

System - Tx/Rx

Frequency 1H	63.680140 MHz
? Ref. Amplitude 1H	0.000 V
Reset	Off
Image Scaling	1.000

Physio - Signal

1st Signal/Mode	None
TR	13.0 ms
Segments	1
Concatenations	18

Physio - Cardiac

Tagging	None
Fat-Water Contrast	Standard
Magn. Preparation	None
Dark Blood	Off

Physio - Cardiac

FoV Read	300 mm
FoV Phase	100.0 %
Phase Resolution	50 %
Dynamic Mode	Standard

Physio - PACE

Resp. Control	Off
Concatenations	18

Inline - Liver

Liver Registration	Off
Save Original Images	On

Inline - Subtraction

Subtract	Off
Measurements	1
StdDev	Off
Save Original Images	On

Inline - Cardiac

Magn. Preparation	None
Save Original Images	On
Contrasts	1
TE	7.30 ms
TR	13.0 ms

Inline - MIP

MIP Sag	Off
MIP Cor	Off
MIP Tra	Off
MIP Time	Off
Radial MIP	Off
Save Original Images	On
MPR Sag	Off
MPR Cor	Off
MPR Tra	Off

Inline - Soft Tissue

Wash-in	Off
Wash-out	Off
TTP	Off
PEI	Off
MIP Time	Off
Measurements	1

Inline - Composing

Inline Composing	Off
------------------	-----

Sequence - Part 1

Sequence Name	fl_rr
Dimension	2D
Excitation	Slice-sel.
RF Pulse Type	Normal
Gradient Mode	Whisper
Flow Compensation	Read
Bandwidth	210 Hz/Px
Asymmetric Echo	Off
Segments	1

Sequence - Part 2

Introduction	On
RF Spoiling	On
Acoustic noise reduction	Off

Sequence - Assistant

SAR Assistant	Off
---------------	-----

\\ZZ_DWI_QIBA\DWI_QIBA\QA_MSC\QA_200622\QIBA_diff_qa_ax

TA: 3:54 min Coil Selection: Auto Voxel Size: 1.7×1.7×4.0 mm³ Acc.: 2 Rel. SNR: 1.00**Properties**

Start measurement without further preparation	Off
Wait for User to Start	Off
Start measurements	Single Measurement
Prio Recon	Off
Auto Open Inline Display	Off
Auto Close Inline Display	Off
Load Images to MR View&GO	On
Auto Store Images	On
Load Images to Stamp Segments	Off
Load Images to Graphic Segments	Off
Graphic segment	Default
Inline Movie	Off

Routine

Slice Group	1
Slices	35
Distance Factor	25 %
Position	R1.4 P17.2 H0.6 mm
Orientation	T > C1.0
Phase Encoding Dir.	A >> P
Phase Oversampling	0 %
FoV Read	220 mm
FoV Phase	100.0 %
Slice Thickness	4.0 mm
TR	8000.0 ms
TE	96.00 ms
Concatenations	1
AutoAlign	---

Contrast - Common

TR	8000.0 ms
TE	96.00 ms
MTC	Off
Magn. Preparation	None
Fat-Water Contrast	Standard
Reconstruction	Magnitude

Contrast - Dynamic

Dynamic Mode	Standard
Delay in TR	0.00 ms

Resolution - Common

FoV Read	220 mm
FoV Phase	100.0 %
Slice Thickness	4.0 mm
Base Resolution	130
Phase Resolution	100 %
Interpolation	Off

Resolution - Acceleration

Acceleration mode	GRAPPA
Reference Scans	EPI/Separate
Acceleration Factor PE	2
Reference Lines PE	40
Phase Partial Fourier	6/8

Resolution - Filter

Raw Filter	On
Elliptical Filter	Off

Resolution - Filter

Distortion Correction	2D
Normalize	Prescan
Noise Masking	Off

Geometry - Common

Slice Group	1
Slices	35
Distance Factor	25 %
Position	R1.4 P17.2 H0.6 mm
Orientation	T > C1.0
Phase Encoding Dir.	A >> P
Phase Oversampling	0 %
FoV Read	220 mm
FoV Phase	100.0 %
Slice Thickness	4.0 mm
TR	8000.0 ms
Multi-Slice Mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice Group	1
Position	R1.4 P17.2 H0.6 mm
Orientation	T > C1.0
Phase Encoding Dir.	A >> P
AutoAlign	---
Initial Position	R1.4 P17.2 H0.6
R	1.4 mm
P	17.2 mm
H	0.6 mm
Initial Orientation	T > C
T > C	1.00
> S	0.00
Initial Rotation	0.00 deg

Geometry - Navigator**Geometry - Saturation**

Special Saturation	None
--------------------	------

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table Position	0 mm
Table Position	H
Inline Composing	Off

System - Miscellaneous

Coil Selection	Auto Coil Select
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combination	Adaptive Combine
Matrix Optimization	Off
Coil Focus	Flat

System - Adjustments

Adjustment Strategy	Standard
B0 Shim	Standard
CoilShim	Off

System - Adjustments

Adjustment Tolerance	Auto
Adjust with Body Coil	Off
Confirm Frequency	Never
Assume Silicone	Off

System - Adjust Volume

Position	R1.4 P17.2 H0.6 mm
Orientation	T > C1.0
Rotation	0.00 deg
A >> P	220 mm
R >> L	220 mm
F >> H	174 mm
Reset	Off

System - Tx/Rx

Frequency 1H	63.680140 MHz
? Ref. Amplitude 1H	0.000 V
Reset	Off
Image Scaling	1.000

Physio - Signal

1st Signal/Mode	None
TR	8000.0 ms
Concatenations	1

Physio - PACE

Resp. Control	Off
Concatenations	1

Diff

Diffusion Mode	3-Scan Trace
Diff. Directions	3
Diffusion Scheme	Bipolar
Diff. Weightings	5
b-value 1	0 s/mm ²
b-value 2	500 s/mm ²
b-value 3	1000 s/mm ²
b-value 4	1500 s/mm ²
b-value 5	2000 s/mm ²
Averages 1	2
Averages 2	2
Averages 3	2
Averages 4	2
Averages 5	2
Dynamic Field Correction	Off
Invert Gray Scale	Off
Diff. Weighted Images	Off
Trace Weighted Images	On
Tensor	Off
FA Maps	Off
ADC Maps	On
Exponential ADC Maps	Off
b-value >=	0 s/mm ²
ADC Noise Threshold	40
Noise Masking	Off
Calculated Image	Off

Sequence - Part 1

Sequence Name	epse
Excitation	Standard
RF Pulse Type	Normal
Gradient Mode	Fast
Bandwidth	1132 Hz/Px
Echo Spacing	0.97 ms

Sequence - Part 1

Free Echo Spacing	Off
Optimization	None
EPI Factor	130

Sequence - Part 2

Introduction	On
Phase Correction	Internal

\ZZ_DWI_QIBA\DWI_QIBA\QA_MSC\QA_200622\localizer

TA: 31 sec Coil Selection: Auto Voxel Size: 1.2×1.2×10.0 mm³ Acc.: None Rel. SNR: 1.00**Properties**

Start measurement without further preparation	Off
Wait for User to Start	Off
Start measurements	Single Measurement
Prio Recon	Off
Auto Open Inline Display	Off
Auto Close Inline Display	Off
Load Images to MR View&GO	On
Auto Store Images	On
Load Images to Stamp Segments	On
Load Images to Graphic Segments	On
Graphic segment	Default
Inline Movie	Off

Routine

Slice Group	1
Slices	1
Distance Factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Slice Group	2
Slices	1
Distance Factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Slice Group	3
Slices	1
Distance Factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Phase Oversampling	0 %
FoV Read	300 mm
FoV Phase	100.0 %
Slice Thickness	10.0 mm
TR	13.0 ms
TE	7.30 ms
Averages	1
Concatenations	18
AutoAlign	---

Contrast - Common

TR	13.0 ms
TE	7.30 ms
TD	0.00 ms
MTC	Off
Magn. Preparation	None
Flip Angle	40 deg
Fat-Water Contrast	Standard
Dark Blood	Off
Contrasts	1
Reconstruction	Magnitude

Contrast - Dynamic

Dynamic Mode	Standard
Measurements	1
Multiple Series	Each Measurement

Resolution - Common

FoV Read	300 mm
FoV Phase	100.0 %
Slice Thickness	10.0 mm
Base Resolution	256
Phase Resolution	50 %
Interpolation	Off

Resolution - Acceleration

Acceleration mode	None
Phase Partial Fourier	Off
Asymmetric Echo	Off

Resolution - Filter

Raw Filter	Off
Elliptical Filter	Off
Distortion Correction	2D
Normalize	Off
Image Filter	Off

Geometry - Common

Slice Group	1
Slices	1
Distance Factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Slice Group	2
Slices	1
Distance Factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Slice Group	3
Slices	1
Distance Factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Phase Oversampling	0 %
FoV Read	300 mm
FoV Phase	100.0 %
Slice Thickness	10.0 mm
TR	13.0 ms
Multi-Slice Mode	Sequential
Series	Interleaved
Concatenations	18

Geometry - AutoAlign

Slice Group	1
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Slice Group	2
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Slice Group	3
Position	Isocenter
Orientation	Sagittal
Phase Encoding Dir.	A >> P

Geometry - AutoAlign

AutoAlign	---
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Orientation	Sagittal
Initial Rotation	0.00 deg

Geometry - Saturation

Saturation Mode	Standard
Special Saturation	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table Position	0 mm
Table Position	H
Inline Composing	Off

System - Miscellaneous

Coil Selection	Auto Coil Select
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combination	Adaptive Combine
Matrix Optimization	Off
Coil Focus	Flat

System - Adjustments

Adjustment Strategy	Standard
B0 Shim	Tune up
CoilShim	Off
Adjustment Tolerance	Auto
Adjust with Body Coil	Off
Confirm Frequency	Never
Assume Silicone	Off

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

System - Tx/Rx

Frequency 1H	63.680140 MHz
? Ref. Amplitude 1H	0.000 V
Reset	Off
Image Scaling	1.000

Physio - Signal

1st Signal/Mode	None
TR	13.0 ms
Segments	1
Concatenations	18

Physio - Cardiac

Tagging	None
Fat-Water Contrast	Standard
Magn. Preparation	None
Dark Blood	Off

Physio - Cardiac

FoV Read	300 mm
FoV Phase	100.0 %
Phase Resolution	50 %
Dynamic Mode	Standard

Physio - PACE

Resp. Control	Off
Concatenations	18

Inline - Liver

Liver Registration	Off
Save Original Images	On

Inline - Subtraction

Subtract	Off
Measurements	1
StdDev	Off
Save Original Images	On

Inline - Cardiac

Magn. Preparation	None
Save Original Images	On
Contrasts	1
TE	7.30 ms
TR	13.0 ms

Inline - MIP

MIP Sag	Off
MIP Cor	Off
MIP Tra	Off
MIP Time	Off
Radial MIP	Off
Save Original Images	On
MPR Sag	Off
MPR Cor	Off
MPR Tra	Off

Inline - Soft Tissue

Wash-in	Off
Wash-out	Off
TTP	Off
PEI	Off
MIP Time	Off
Measurements	1

Inline - Composing

Inline Composing	Off
------------------	-----

Sequence - Part 1

Sequence Name	fl_rr
Dimension	2D
Excitation	Slice-sel.
RF Pulse Type	Normal
Gradient Mode	Whisper
Flow Compensation	Read
Bandwidth	210 Hz/Px
Asymmetric Echo	Off
Segments	1

Sequence - Part 2

Introduction	On
RF Spoiling	On
Acoustic noise reduction	Off

Sequence - Assistant

SAR Assistant	Off
---------------	-----

\\ZZ_DWI_QIBA\DWI_QIBA\QA_MSC\QA_200622\qiba_diff_qa_sag

TA: 3:54 min Coil Selection: Auto Voxel Size: 1.7×1.7×4.0 mm³ Acc.: 2 Rel. SNR: 1.00**Properties**

Start measurement without further preparation	Off
Wait for User to Start	Off
Start measurements	Single Measurement
Prio Recon	Off
Auto Open Inline Display	Off
Auto Close Inline Display	Off
Load Images to MR View&GO	On
Auto Store Images	On
Load Images to Stamp Segments	Off
Load Images to Graphic Segments	Off
Graphic segment	Default
Inline Movie	Off

Routine

Slice Group	1
Slices	35
Distance Factor	25 %
Position	R1.4 P17.2 H0.6 mm
Orientation	Sagittal
Phase Encoding Dir.	H >> F
Phase Oversampling	0 %
FoV Read	220 mm
FoV Phase	100.0 %
Slice Thickness	4.0 mm
TR	8000.0 ms
TE	96.00 ms
Concatenations	1
AutoAlign	---

Contrast - Common

TR	8000.0 ms
TE	96.00 ms
MTC	Off
Magn. Preparation	None
Fat-Water Contrast	Standard
Reconstruction	Magnitude

Contrast - Dynamic

Dynamic Mode	Standard
Delay in TR	0.00 ms

Resolution - Common

FoV Read	220 mm
FoV Phase	100.0 %
Slice Thickness	4.0 mm
Base Resolution	130
Phase Resolution	100 %
Interpolation	Off

Resolution - Acceleration

Acceleration mode	GRAPPA
Reference Scans	EPI/Separate
Acceleration Factor PE	2
Reference Lines PE	40
Phase Partial Fourier	6/8

Resolution - Filter

Raw Filter	On
Elliptical Filter	Off

Resolution - Filter

Distortion Correction	2D
Normalize	Prescan
Noise Masking	Off

Geometry - Common

Slice Group	1
Slices	35
Distance Factor	25 %
Position	R1.4 P17.2 H0.6 mm
Orientation	Sagittal
Phase Encoding Dir.	H >> F
Phase Oversampling	0 %
FoV Read	220 mm
FoV Phase	100.0 %
Slice Thickness	4.0 mm
TR	8000.0 ms
Multi-Slice Mode	Interleaved
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slice Group	1
Position	R1.4 P17.2 H0.6 mm
Orientation	Sagittal
Phase Encoding Dir.	H >> F
AutoAlign	---
Initial Position	R1.4 P17.2 H0.6
R	1.4 mm
P	17.2 mm
H	0.6 mm
Initial Orientation	Sagittal
Initial Rotation	90.00 deg

Geometry - Navigator**Geometry - Saturation**

Special Saturation	None
--------------------	------

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table Position	0 mm
Table Position	H
Inline Composing	Off

System - Miscellaneous

Coil Selection	Auto Coil Select
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combination	Adaptive Combine
Matrix Optimization	Off
Coil Focus	Flat

System - Adjustments

Adjustment Strategy	Standard
B0 Shim	Standard
CoilShim	Off
Adjustment Tolerance	Auto
Adjust with Body Coil	Off

System - Adjustments

Confirm Frequency	Never
Assume Silicone	Off

System - Adjust Volume

Position	R1.4 P17.2 H0.6 mm
Orientation	Sagittal
Rotation	90.00 deg
F >> H	220 mm
A >> P	220 mm
R >> L	174 mm
Reset	Off

System - Tx/Rx

Frequency 1H	63.680140 MHz
? Ref. Amplitude 1H	0.000 V
Reset	Off
Image Scaling	1.000

Physio - Signal

1st Signal/Mode	None
TR	8000.0 ms
Concatenations	1

Physio - PACE

Resp. Control	Off
Concatenations	1

Diff

Diffusion Mode	3-Scan Trace
Diff. Directions	3
Diffusion Scheme	Bipolar
Diff. Weightings	5
b-value 1	0 s/mm ²
b-value 2	500 s/mm ²
b-value 3	1000 s/mm ²
b-value 4	1500 s/mm ²
b-value 5	2000 s/mm ²
Averages 1	2
Averages 2	2
Averages 3	2
Averages 4	2
Averages 5	2
Dynamic Field Correction	Off
Invert Gray Scale	Off
Diff. Weighted Images	Off
Trace Weighted Images	On
Tensor	Off
FA Maps	Off
ADC Maps	On
Exponential ADC Maps	Off
b-value >=	0 s/mm ²
ADC Noise Threshold	40
Noise Masking	Off
Calculated Image	Off

Sequence - Part 1

Sequence Name	epse
Excitation	Standard
RF Pulse Type	Normal
Gradient Mode	Fast
Bandwidth	1132 Hz/Px
Echo Spacing	0.97 ms
Free Echo Spacing	Off
Optimization	None

Sequence - Part 1

EPI Factor	130
------------	-----

Sequence - Part 2

Introduction	On
Phase Correction	Internal

\ZZ_DWI_QIBA\DWI_QIBA\QA_MSC\QA_200622\QIBA_qa_vibe_sag_iso

TA: 2:43 min Coil Selection: Auto Voxel Size: 1.0×1.0×1.0 mm³ Acc.: 2 Rel. SNR: 1.00**Properties**

Start measurement without further preparation	On
Wait for User to Start	Off
Start measurements	Single Measurement
Prio Recon	Off
Auto Open Inline Display	Off
Auto Close Inline Display	Off
Load Images to MR View&GO	On
Auto Store Images	On
Load Images to Stamp Segments	On
Load Images to Graphic Segments	Off
Graphic segment	Default
Inline Movie	Off

Routine

Slab Group	1
Slabs	1
Distance Factor	20 %
Position	R1.4 P18.6 H0.7 mm
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Slices per Slab	192
Phase Oversampling	0 %
Slice Oversampling	0.0 %
FoV Read	250 mm
FoV Phase	100.0 %
Slice Thickness	1.0 mm
TR	6.0 ms
TE	2.38 ms
Averages	1
Concatenations	1
AutoAlign	Head > Orbits

Contrast - Common

TR	6.0 ms
TE	2.38 ms
Flip Angle	10 deg
Fat-Water Contrast	Standard
Contrasts	1
Reconstruction	Magnitude

Contrast - Dynamic

Dynamic Mode	Standard
Temporal Interpolation	1
Measurements	1
Multiple Series	Off
3D Reordering	Standard
Time to Center	82.6 s
Burn Time to Center	Off

Resolution - Common

FoV Read	250 mm
FoV Phase	100.0 %
Slice Thickness	1.0 mm
Base Resolution	256
Phase Resolution	100 %
Slice Resolution	100 %
Trajectory	Cartesian
Interpolation	Off

Resolution - Acceleration

Acceleration mode	GRAPPA
Total Factor	2
Reference Scans	Integrated
Acceleration Factor PE	2
Reference Lines PE	24
Acceleration Factor 3D	1
Phase Partial Fourier	Off
Slice Partial Fourier	Off
Asymmetric Echo	Weak
Elliptical Scanning	Off

Resolution - Filter

Raw Filter	Off
Elliptical Filter	On
POCS	Off
Distortion Correction	2D
Normalize	Prescan
Noise Masking	Off
Image Filter	Off

Geometry - Common

Slab Group	1
Slabs	1
Distance Factor	20 %
Position	R1.4 P18.6 H0.7 mm
Orientation	Sagittal
Phase Encoding Dir.	A >> P
Slices per Slab	192
Phase Oversampling	0 %
Slice Oversampling	0.0 %
FoV Read	250 mm
FoV Phase	100.0 %
Slice Thickness	1.0 mm
TR	6.0 ms
Series	Ascending
Concatenations	1

Geometry - AutoAlign

Slab Group	1
Position	R1.4 P18.6 H0.7 mm
Orientation	Sagittal
Phase Encoding Dir.	A >> P
AutoAlign	Head > Orbits
Initial Position	R1.4 P18.6 H0.7
R	1.4 mm
P	18.6 mm
H	0.7 mm
Initial Orientation	Sagittal
Initial Rotation	0.00 deg

Geometry - Saturation

Special Saturation	None
--------------------	------

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table Position	0 mm
Table Position	H
Inline Composing	Off

System - Miscellaneous

Coil Selection	Auto Coil Select
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combination	Adaptive Combine
Matrix Optimization	Off
Coil Focus	Flat

System - Adjustments

Adjustment Strategy	Standard
B0 Shim	Standard
CoilShim	Off
Adjustment Tolerance	Auto
Adjust with Body Coil	Off
Confirm Frequency	Never
Assume Silicone	Off

System - Adjust Volume

Position	R1.4 P18.6 H0.7 mm
Orientation	Sagittal
Rotation	0.00 deg
A >> P	250 mm
F >> H	250 mm
R >> L	192 mm
Reset	Off

System - Tx/Rx

Frequency 1H	63.680140 MHz
? Ref. Amplitude 1H	0.000 V
Reset	Off
Image Scaling	1.000

Physio - PACE

Resp. Control	Off
Concatenations	1

Inline - Liver

Liver Registration	Off
Save Original Images	On

Inline - Subtraction

Subtract	Off
Measurements	1
StdDev	Off
Save Original Images	On

Inline - Cardiac

Save Original Images	On
Contrasts	1
TE	2.38 ms
TR	6.0 ms

Inline - MIP

MIP Sag	Off
MIP Cor	Off
MIP Tra	Off
MIP Time	Off
Radial MIP	Off
Save Original Images	On
MPR Sag	Off
MPR Cor	Off
MPR Tra	Off

Inline - Soft Tissue

Wash-in	Off
Wash-out	Off
TTP	Off
PEI	Off
MIP Time	Off
Measurements	1

Inline - Composing

Inline Composing	Off
------------------	-----

Sequence - Part 1

Sequence Name	fl
Dimension	3D
Excitation	Slab-sel.
RF Pulse Type	Normal
Gradient Mode	Normal
Bandwidth	810 Hz/Px
Asymmetric Echo	Weak
Optimization	None

Sequence - Part 2

Introduction	On
RF Spoiling	On
Incr. Gradient Spoiling	On
Breast Application	Off

Sequence - Assistant

SAR Assistant	Off
Optimization	None