

<b>MIRACLES-GGC-FORM-002</b>	<b>NHS Greater Glasgow &amp; Clyde</b>
<b>Checklist</b>	

Participant ID.....

Date.....

Initial

Scan prep

Ensure Hyperfine is plugged in. Turn into scan mode (refer to MIRACLES-GGC-SOP-001)	
Safety questionnaire completed for patient and safe to scan (refer to MIRACLES-GGC-SOP-002)	
Check patient for metallic objects and remove	
Trolley check for potential unsafe objects and remove	
Ensure oxygen cylinder secure under trolley	
Any monitoring equipment placed to the side of the scanner	
All staff entering the MR Environment authorised. Check all metallic objects removed	

Prior to scan start

Earplugs/ side pads fitted	
Room door locked during operation if possible	

After the scan

Images sent to ORTHANC (section 5, MIRACLES-GGC-SOP-002)	
Clear data on the patient tab	
Switch off only after all data has been uploaded	
Clean the Swoop system	
Replace barrier	
Return key and ipad	
Ensure all documents, including this checklist, are given to the MIRACLES study team	

Author	Owner	Revision	Active Date	Review date	Page
TH	KM	1	10/11/23	As required	1
This document is uncontrolled when printed. Check Revision BEFORE use!					